

# OSSIX<sup>®</sup> Volumax

Correction of Buccal Ridge Deficiency  
at Implant Second Stage Surgery

Dr. Rodrigo Neiva, Gainesville, FL


## Case background

32 year old female presented with tooth #4 missing. Tooth had been extracted 5 years before initial examinations. A 3.6x13 mm implant was placed in conjunction with GBR to correct horizontal alveolar bone deficiency. At second stage surgery, volumetric soft tissue deficiency and thin buccal plate were found. Soft and hard tissue augmentation were performed using a double-layered OSSIX® Volumax scaffold, without bone grafting materials.


## Pre-Op



*Occlusal view of #4 implant before surgery to uncover the implant. Soft tissue volume deficiency detected*

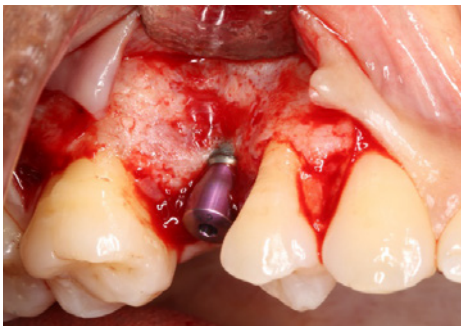

*Flap elevation showing that the buccal plate was thin, but still present*

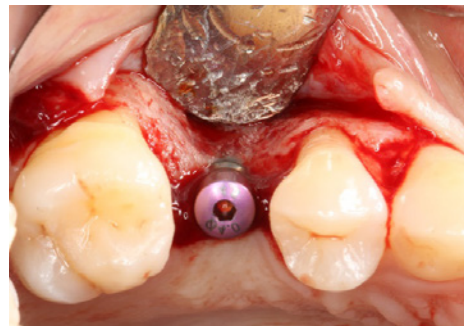

*Occlusal view illustrates ridge deficiency that would result in poor esthetic outcome of the future implant restoration*


## Procedure




*OSSIX Volumax (25x30mm) after hydration in sterile saline*




*OSSIX Volumax folded in half for additional tenting and tissue thickness*

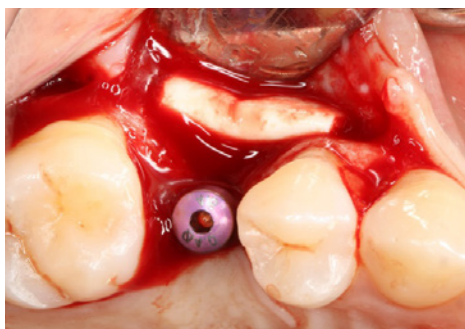

*OSSIX Volumax adapted for optimal ridge without bone graft*

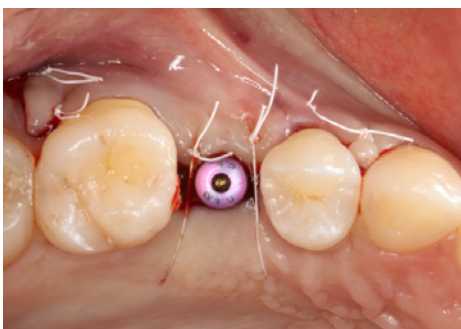

*Buccal flag positioned and sutured with 4.0 PTFE sutures*

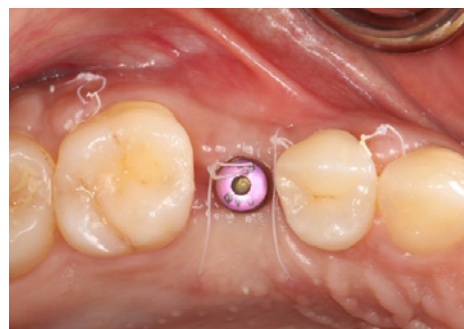


*2-week post-op showing optimal wound healing and ridge contours*

Please refer to OSSIX Volumax IFU for the full information on indications and contraindications.



*Provisional restoration was installed at 30 days to develop soft tissue profile*

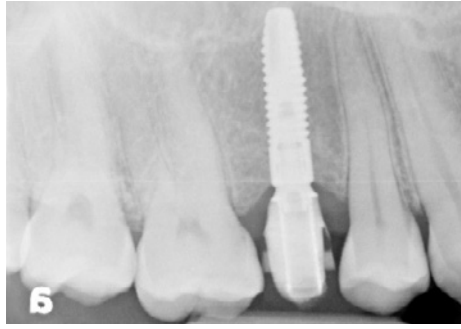


*Radiograph at 30 days showing the provisional restoration*


### Follow-up – 3 to 5 months



*Occlusal view at 3 months before final impressions*

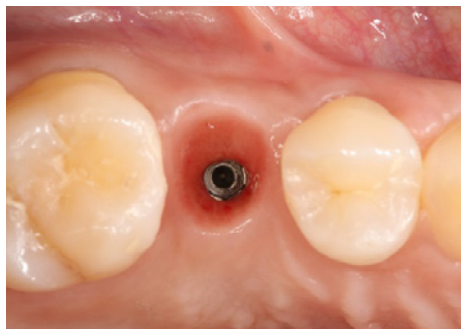

*Occlusal view at 5 months showing stable results and soft tissue health*

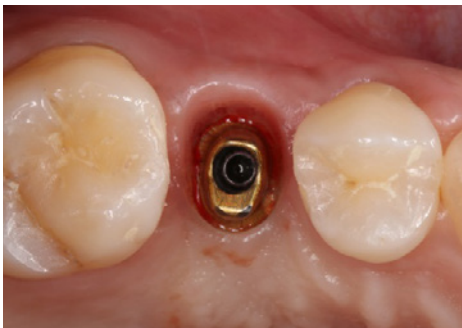


*Abutment adaptation to the developed soft tissue profile*



*Final implant-supported crown – 5 months*

### Follow-up – 6 months

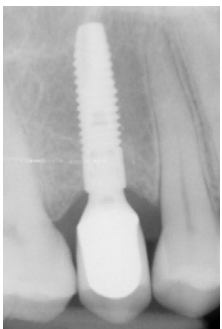

*Radiograph at 6 months showing optimal crestal bone levels*



*Follow-up at 6 months showing stable results after final restoration*



*Buccal view showing excellent esthetic result with OSSIX Volumax*



*6 months CT scan. Original bone and OSSIX Volumax ossification visible*

# Correction of Buccal Ridge Deficiency at Implant Second Stage Surgery

---

OSSIX® Volumax is a thick, cross-linked, ossifying collagen scaffold, which restores lost volume in guided bone regeneration (GBR) and guided tissue regeneration (GTR) procedures.

---

OSSIX Volumax is available in 4 sizes.

OSSIX Volumax is currently available in the USA, Europe, Argentina, Canada, Chile, Colombia, Dominican Republic, Hong Kong, India, Israel, Korea, Mexico, and Singapore.

datumdental



[www.ossixdental.com](http://www.ossixdental.com)